
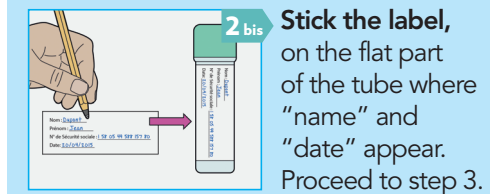
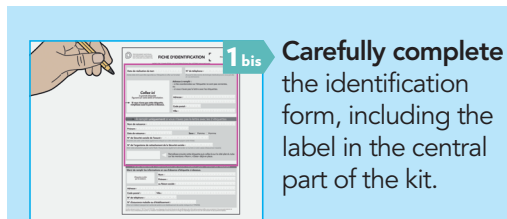
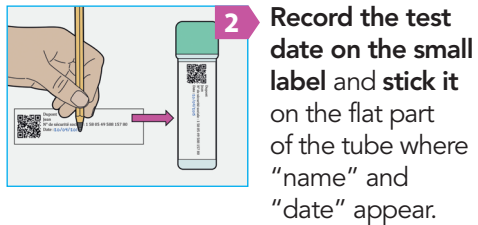







INSTRUCTIONS

If you have a letter inviting you to do the test with 2 labels, follow steps 1 and 2, then go to step 3.

If you do not have this letter, go straight to step 1bis below.

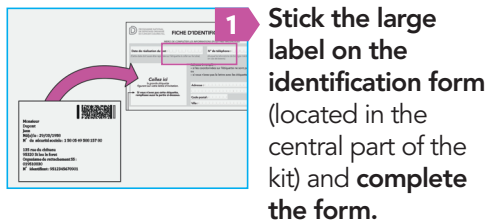

IMPORTANT: for the test to work, there must be no contact with any liquid, so no urine, no bleach, nothing.

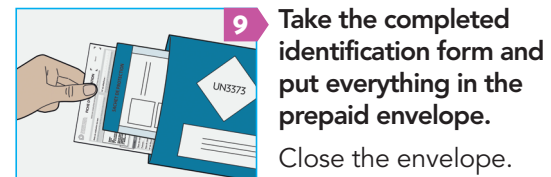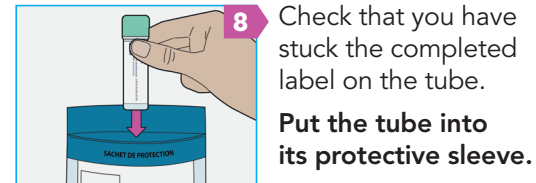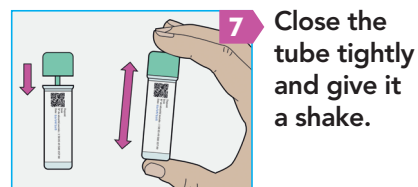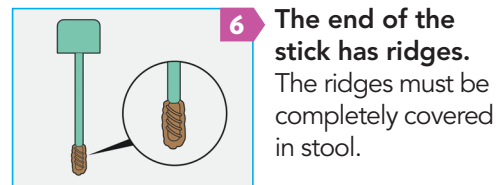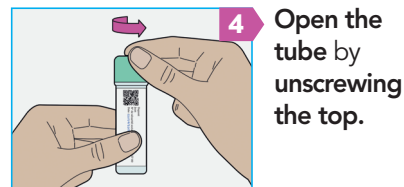

You should receive your results by post within two weeks.

For online results, please register at the following website: www.resultat-depistage.fr

FOR FURTHER INFORMATION?

- Ask your doctor.
- On line at: www.e-cancer.fr, "Dépistage/Screening" section.
- Call: **0 805 123 124** (local call cost) Monday-Friday, 9:00 am-7:00 pm and Saturday, 9:00 am-2:00 pm.
- Contact the department responsible for screening in your area or on e-cancer.fr.

FAQ

- **Within what period of time must the test be used?**
The expiry date is on the tube.
- **What if the tube gets lost or damaged?**
Ask your doctor for a new test.
- **Something went wrong with the sample collection, what should I do?**
Contact your doctor; he will advise accordingly you and give you a new test.